## AccuFit® Sentinelle® Coil Drape

REF SCD301 (Patent #USD622,389 S)

Single use only  
Para un solo uso.  
A usage unique (jetable) seulement

NON STERILE

LATEX FREE

Medline Industries Inc.  
Manufactured exclusively for:  
QSUM Biopsy Disposables LLC.  
PO Box 2581, Baxter, MN 56425  
877-304-4446 Toll Free (218)297-2012 Fax  
[www.qsum.net](http://www.qsum.net) Made in Thailand

### English - en

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## English- en

2. Tuck in the area around the pads and drop the elastic through the coil holes. Make sure the holes are centered.

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RD12PXS.

### English- en

3. Take elastic and attach around the arched arm and attach to the Velcro strip on the drape. Do the same on the other 3 elastic pieces making sure the drape lines up with the holes.

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RD12PXS.

### English- en

*NOTE: If you are not using the side pads, wrap elastic around the arched side arms twice and secure as shown. Do this only on one elastic piece so the drape can be easily pulled to the side until the patient has been positioned. The drape should be pulled back so the patient has access to grip the arched white side bar until she is in place.*

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RD12PXS.

## English- en

5. Tuck in the area where the bottom of the breast would sit under so the drape covers the entire pad. The material will automatically adhere to the Velcro that the manufacturer's padding attaches to. Tuck this under all the way across the bottom of the coil holes.

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RD12PXS.

**English- en**

6. View of finished draping for the “stand alone” and “full coil table”.

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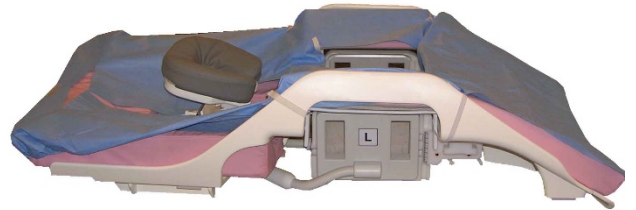
\*AccuFit and QSUM are registered trademarks of QSUM.

**QSUM Biopsy Disposables LLC**

P.O. Box 2581 Baxter, MN 56425

218.270.3533 Office \* 218.297.2012 Fax \* 877.304.4446 toll free

\* [www.qsum.net](http://www.qsum.net)

\* e-mail: [Qsumservice@qsum.net](mailto:Qsumservice@qsum.net)

RD12PXS.