AccuFit® Stage Draping System Instructions For the Hologic/Lorad® MultiCare™ Breast Biopsy System

Single use only
Para un solo uso.
À usage unique (jetable) seulement

REF

LS701 (Patent #US7,775,213 B2 & #US 8,196,586 B2)

NON-STERILE

LATEX FREE

Medline Industries Inc.
Manufactured exclusively for:
QSUM Biopsy Disposables LLC.
PO Box 2581, Baxter, MN 56425
877-304-4446 Toll Free (218)297-2012 Fax
www.qsum.net Made in China

1. English- en

The following should be in the package:

- Stage Drape
- Needle Guidance Drape
- Control Module Cover

First install the control module cover (#1 & #2)

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RD12PXS

2. English- en

The next piece is the stage drape. Start at the camera. The camera end has the long white strip across the top of the plastic (#3). Pull the plastic tight over the camera face. Remove the two sticky tabs and adhere (#4).

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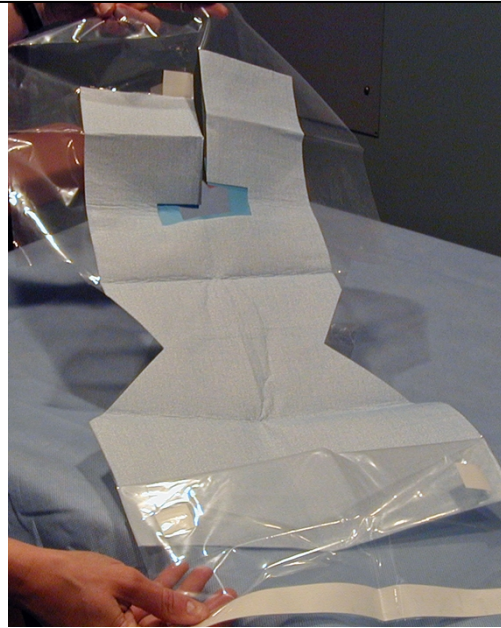
RD12PXS

3. English- en

Pull drape through the paddle hole. (#5 & #6)
Use serrations to make the drape fit the smaller paddle.

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#6

4. English- en

Bring the drape under the needle guidance arm. For a secure fit, the rubber boot goes around the stage unit center support post. (#7)

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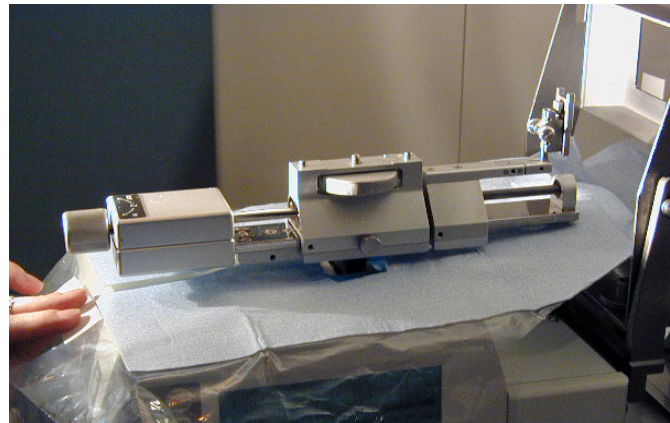
RD12PXS

5. English- en

Move the needle guidance forward. Small opening goes over the needle guide holder. (#8) Remove the sticky tab and close the opening behind the needle guide for a tight fit. The large opening in the rubber will go over the screw.

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#8

6. English- en

Image shows completed installation. (#9)

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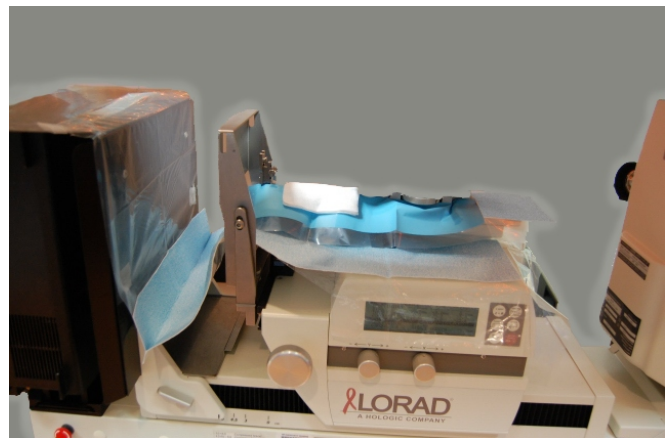
9

*AccuFit and QSUM are registered trademarks of QSUM Biopsy Disposables LLC. Lorad and MultiCare are registered trademarks of Hologic.

QSUM Biopsy Disposables LLC

P.O. Box 2581 Baxter, MN 56425

218.270.3533 Office * 218.297.2012 Fax * 877.304.4446 toll free

* www.qsum.net

* e-mail: Qsumservice@qsum.net

RD12PXS