
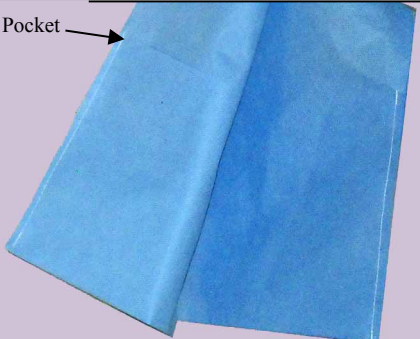






Disposable Mastectomy Drain Scarf Product Instructions P/N MS401



Medline Industries Inc.
Manufactured exclusively for:
QSUM Biopsy Disposables LLC.
PO Box 2581, Baxter, MN 56425
877-304-4446 Toll Free (218)297-2012 Fax
www.qsum.net Made in Mexico

<p>1. Open package and remove a scarf. Place around neck as show in FIG. 1. NOTE: the pockets should face in toward the body as shown in FIG 1.</p>	 <p>FIG. 1</p>
<p>2. Place a drain in each pocket. (see FIG. 2) If the patient has an odd number of drains make sure something of equal weight is placed in the opposite pocket. This is to offset any problem with the drain tube pulling on the insertion point.</p>	 <p>FIG. 2</p>
<p>3. To remove a pocket, or both pockets, pull apart the Velcro. Make sure the “soft” side of the Velcro is facing the patient’s clothing. (FIG.3)</p>	 <p>FIG. 3</p>
<p>4. Change scarf if bodily fluids leak from the drains and dispose. Wear under clothing or use in the shower and hang up to dry.</p>	

QSUM is a registered trademark.

QSUM Biopsy Disposables LLC

P.O. Box 2581 Baxter, MN 56425

218.270.3533 Office * 218.297.2012 Fax * 877.304.4446 toll free

* www.qsum.net

* e-mail: Qsumservice@qsum.net



RD12PXS.