AccuFit® Hologic Affirm® Tomo/Stereo Stage Draping

P/N: LS703A

REF **LS703A** (Patent #US7,775,213 B2)

2 Single use only
Para un solo uso.
À usage unique (jetable) seulement

NON-STERILE LATEX FREE  **CE**

 Medline Industries Inc.
Manufactured exclusively for:
QSUM Biopsy Disposables LLC.
PO Box 2581, Baxter, MN 56425
(218) 270-3533 www.qsum.net Made in China

English- en

1. The following should be in the package:

- a. Stage Drape
- b. Control Module Cover
- c. Paddle Retention Bag
- d. Needle Guidance Drape

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English- en

2. First install the control module cover.

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English- en

3. Place the plastic biopsy paddle in the small clear fluid retention bag.

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English- en

4. Open drape and start at the end with the blue slit and place around the pillar. Lay the drape over where the paddle will attach.

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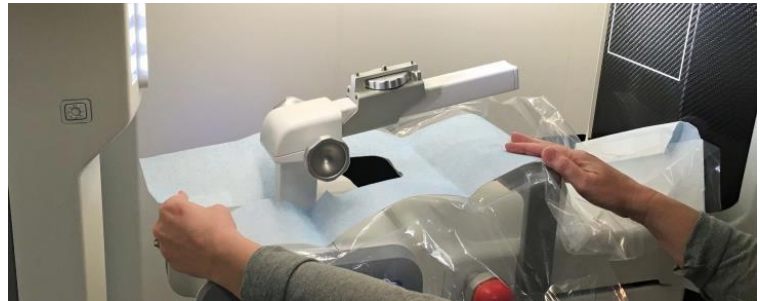
English- en

5. Locate the "Place Paddle Here" label and note the small cut out. Below is a piece of sticky tape. Remove the sticky tape and attach it to the paddle frame. Make sure the metal tab is in the cut out area.

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English- en

6. Remove white sticky tape and place along the top of the camera. The plastic completely covers the face of the camera. Remove sticky tabs on both sides of the camera plastic. Pull the plastic tight over the camera to eliminate any wrinkles.

NOTE: If you do not want the plastic to cover the camera, remove at the perforations and attach the sticky tabs.

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English- en

7. Grab the paddle in the fluid retention bag and press the lever to snap it on. Make sure it is tight.

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8. English- en

Move the needle guidance forward. Small opening goes over the needle guide holder. (#8) Remove the sticky tab and close the opening behind the needle guide for a tight fit. The large opening in the rubber will go over the screw.

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