



Post-Surgical Drain Holder Product Instructions P/N MS402



Medline Industries Inc.
Manufactured exclusively for:
QSUM Biopsy Disposables LLC.
PO Box 2581, Baxter, MN 56425
218-270-3533 www.qsum.net Made in China

1. Open package and remove a drain holder. Place around neck as show in FIG. 1.
NOTE: there are 2 pockets in each side; after use in shower, hang to dry. If using under clothing use a safety pin to secure to clothing if motion might disturb the drains.

FIG. 1



2. Place a drain in each pocket. (see FIG. 2)
If the patient has an odd number of drains make sure something of equal weight is placed in the opposite pocket. This is to offset any problem with the drain tube pulling on the insertion point.

FIG. 2



3. Change scarf if bodily fluids leak from the drains and dispose. Wear under clothing or use in the shower and hang up to dry.

QSUM Biopsy Disposables LLC

P.O. Box 2581 Baxter, MN 56425

218.270.3533 Office * 218.297.2012 Fax * 877.304.4446 toll free * www.qsum.net * e-mail: Qsumservice@qsum.net Rev. 5-17

